


# KODAK

## Dental Intraoral E-Speed Film

High-speed for brilliant clarity with reduced radiation exposure

You don't have to make a choice between quality and speed. Kodak Dental Intraoral E-Speed Film provides brilliant image clarity with as little as one-half the exposure time required for traditional D-speed intraoral films.

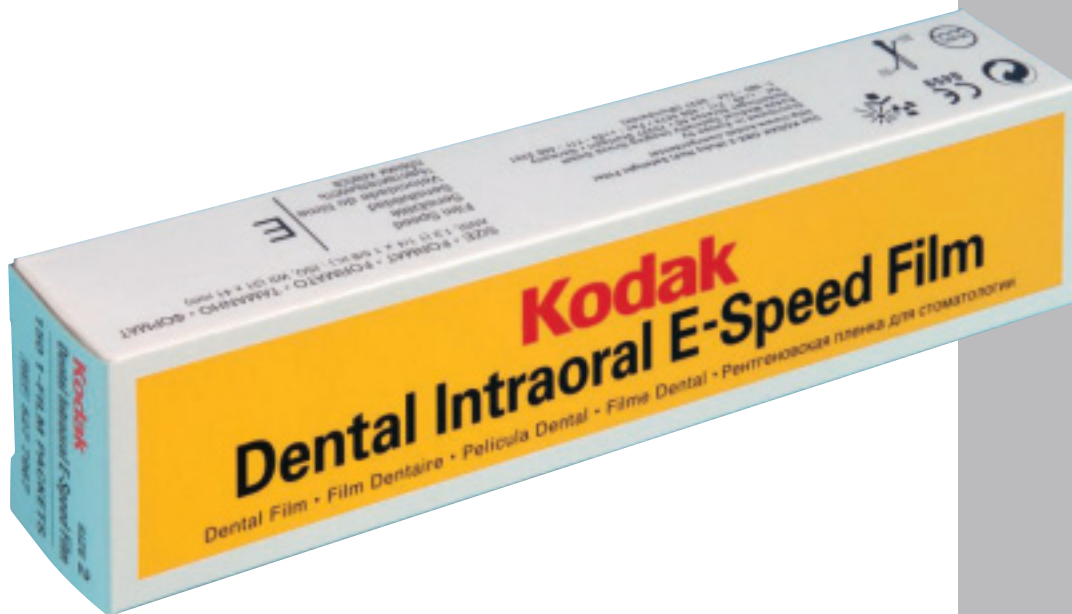

- No loss of sharp, radiographic detail thanks to patented Kodak T-Grain emulsion technology
- Less radiation exposure for patients and staff
- Helps you put patients at ease and minimises fear of X-rays
- Potentially less retakes due to patient movement
- Processes in automatic or manual equipment, so no change in your current processing procedures or equipment is necessary
- Ideal for intraoral exams – no need to order different speed films
- Color-coded vinyl packets are easy to open in the dark
- Conveniently packaged in 150-count boxes



# Recommended exposure settings

KODAK Dental Intraoral 150 E-Speed Film			CAT Number 527 7967									
Settings		kV	50	60	65	65	65	70	70	70	80	
		mA	7	7	7,5	8	10	7	8	10	10	
20 cm	Maxillary	Incisor	+ 40°	0,82	0,33	0,19	0,18	0,14	0,16	0,14	0,12	0,06
		Cuspid	+ 45°	0,82	0,33	0,19	0,18	0,14	0,16	0,14	0,12	0,06
		Bicuspid	+ 30°	1,10	0,44	0,26	0,24	0,19	0,22	0,19	0,15	0,08
		Molar	+ 20°	1,23	0,49	0,29	0,27	0,22	0,25	0,22	0,17	0,09
	Mandibular	Incisor	- 15°	0,69	0,27	0,16	0,15	0,12	0,14	0,12	0,10	0,05
		Cuspid	- 20°	0,69	0,27	0,16	0,15	0,12	0,14	0,12	0,10	0,05
		Bicuspid	- 10°	0,75	0,30	0,18	0,17	0,13	0,15	0,13	0,11	0,05
		Molar	- 5°	0,82	0,33	0,19	0,18	0,14	0,16	0,14	0,12	0,06
	Bite Wing	Anterior (Incisor)	+ 8°	0,69	0,27	0,16	0,15	0,12	0,14	0,12	0,10	0,05
		Posterior (Bicuspid)	+ 8°	0,82	0,33	0,19	0,18	0,14	0,16	0,14	0,12	0,06
		Occlusal	+ 8°	1,37	0,55	0,32	0,30	0,24	0,27	0,24	0,19	0,10
	40 cm	Maxillary	Incisor	+ 40°	3,29	1,32	0,77	0,72	0,58	0,66	0,58	0,46
Cuspid			+ 45°	3,29	1,32	0,77	0,72	0,58	0,66	0,58	0,46	0,23
Bicuspid			+ 30°	4,39	1,76	1,02	0,96	0,77	0,88	0,77	0,61	0,31
Molar			+ 20°	4,94	1,97	1,15	1,08	0,86	0,99	0,86	0,69	0,35
Mandibular		Incisor	- 15°	2,74	1,10	0,64	0,60	0,48	0,55	0,48	0,38	0,19
		Cuspid	- 20°	2,74	1,10	0,64	0,60	0,48	0,55	0,48	0,38	0,19
		Bicuspid	- 10°	3,02	1,21	0,70	0,66	0,53	0,60	0,53	0,42	0,21
		Molar	- 5°	3,29	1,32	0,77	0,72	0,58	0,66	0,58	0,46	0,23
Bite Wing		Anterior (Incisor)	+ 8°	2,74	1,10	0,64	0,60	0,48	0,55	0,48	0,38	0,19
		Posterior (Bicuspid)	+ 8°	3,29	1,32	0,77	0,72	0,58	0,66	0,58	0,46	0,23
		Occlusal	+ 8°	5,49	2,19	1,28	1,20	0,96	1,10	0,96	0,77	0,38

Children: Reduce the exposure time by about 1/3

Edentulous area: Reduce the exposure time by about 1/4

Order Kodak Dental Intraoral E-Speed Film and other supplies today from one convenient source.

Contact your dealer of Kodak Dental Products.



© Eastman Kodak Company, 2004.  
Kodak and T-Grain are trademarks  
of Eastman Kodak Company.  
DoldeMedien 0086\_04

FOR MORE INFORMATION VISIT:  
[www.kodak.com/dental](http://www.kodak.com/dental)

Kodak GmbH · Kodak Dental Systems  
Hedelfinger Strasse 60  
70327 Stuttgart · Germany  
Fax: +49 711 406 3331