



KODAK RP X-OMAT LO Fixer and Replenisher

1) Description

This KODAK packaged fixer consists of two concentrated liquid parts: Part A and Part B. For use, Parts A and B must be mixed together and diluted with water according to instructions provided with the chemicals.

This fixer is recommended for processing KODAK Medical and Dental X-ray films in automatic equipment. It can also be used for manual processing KODAK Medical, Dental, and INDUSTREX Films. Specific directions for using this fixer are provided in TI documents for individual products and process cycles.

2) Mixing Instructions

Instructions for mixing replenisher and/or working solutions are provided by leaflets for individual size packages.

Notice: Observe precautionary information on product labels and on the Material Safety Data Sheets.

3) Storage

The chemicals must be stored and used at 40 to 85°F (4 to 29°C). DO NOT keep chemicals longer than the times below. Discard if there is evidence of contamination, dirt, over-dilution, excessive evaporation, or crystallization.

Concentrate	Length of Time
Full bottles, unopened	2 years
Half filled capped bottles	3 months
Mixed solutions	Approximately 6 weeks ^[1]

^[1]If kept clean and at proper concentration.

4) Replenishment Rates

Automatic Processing

- Recommendations are provided with individual processing TI document.

Manual Processing

1. KODAK Films - Extraoral Dental Applications

- After fixing the equivalent of forty 5 x 12-inch films per 1 gallon (3.8 liters) of fixing bath, remove 1 quart (32 fl oz) (.95 liters) of the used fixing bath.
- Add 1 quart (.95 liters) of working solution mixed according to the instructions packaged with the fixer.
- Stir until the solution is uniform.

2. KODAK Films - Intraoral Dental Applications

- Usually the fixer used for processing intraoral films is not replenished. Discard fixer when clearing time required for the films being processed exceeds 2 minutes.

3. KODAK Medical X-ray Films and KODAK INDUSTREX Films

- After fixing the equivalent of fifty 14 x 17-inch films per 5 gallons (19 liters) of fixing bath, remove 1 gallon (.95 liters) of the used fixer.
- Add 1 gallon (.95 liters) of fixer working solution mixed according to the instructions packaged with the chemicals.
- Stir until a uniform solution is obtained.

Note: If you have questions or need assistance, contact your local Kodak representative.

Note: The contents of this publication are subject to change without notice.

Kodak, X-Omat, and Industrex are trademarks.

Health Imaging
EASTMAN KODAK COMPANY - Rochester, NY 14650

End of Data Sheet
