


## You *can* afford quality Kodak film for general radiography

*Kodak medical x-ray film/general purpose/green brings Kodak film technology to your x-ray department without stretching your budget.*

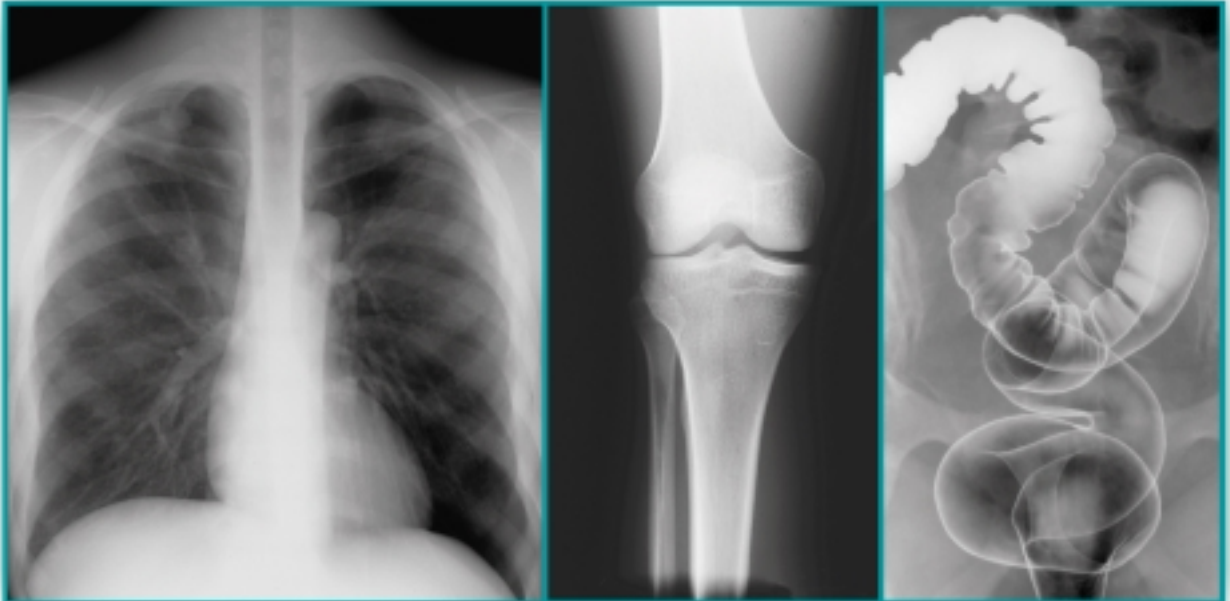

This orthochromatic, medium-contrast film is suitable for all types of radiographic examinations.

Features include:

- Kodak T-Grain® emulsion that delivers high visibility of details without sacrificing speed
- Forgiving exposure and robust processing tolerance to reduce the need for costly re-examinations
- Manual or automatic processing in standard and rapid (60-second) cycles

Quality image—reduced exposure

Kodak medical x-ray film/general purpose/green is part of a family of affordable Kodak offerings for radiography including films, screens, cassettes, film processing chemicals, and automatic film processors.

This orthochromatic film uses Kodak T-Grain technology to deliver improved image resolution, reduced image noise, and higher system speed than most conventional “blue” intensifying screens and films. In addition, the reduced chemical rates recommended for this film can provide chemical cost savings.


Kodak medical x-ray film/general purpose/green is designed for use with economical Kodak green 400 intensifying screens or equivalents. These screens deliver speed and resolution with the imaging power of green-light-emitting rare-earth phosphors.

The resulting screen-film system yields quality images and helps reduce operating expenses.




HEALTH IMAGING  
A BETTER VIEW OF LIFE.



# KODAK MEDICAL X-RAY FILM / GENERAL PURPOSE / GREEN

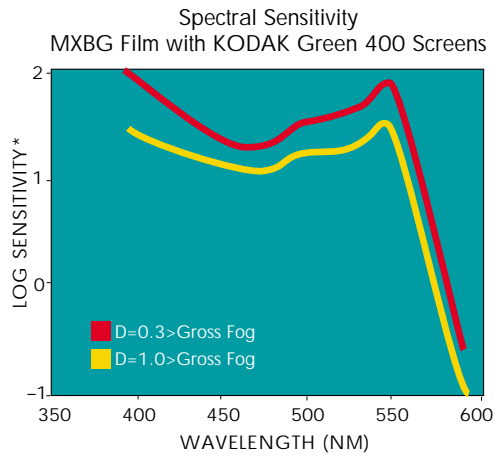

\*Sensitivity=reciprocal of exposure (ergs/sq cm) required to produce specified density

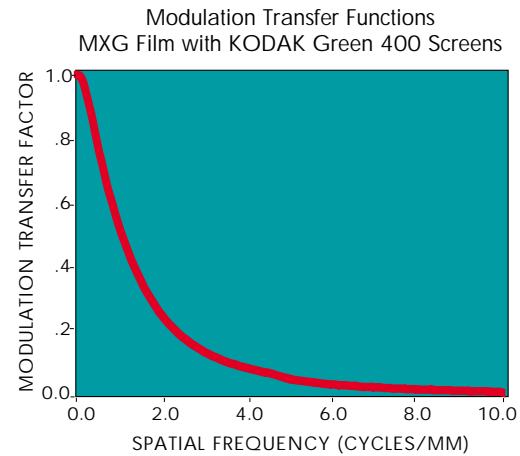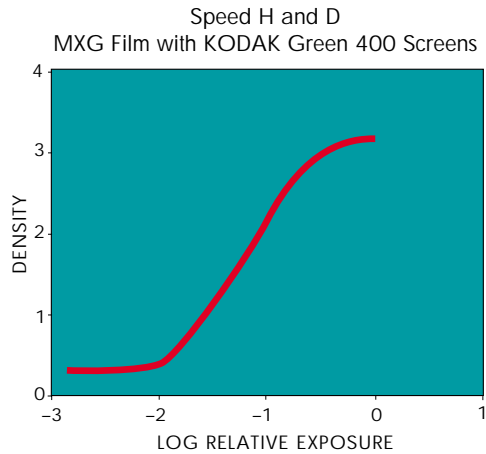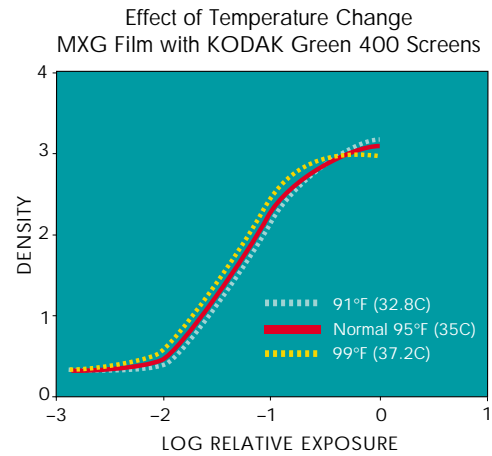

## SPECIFICATIONS

Film sizes	CAT No.
18 x 24 cm	811 6428
24 x 30 cm	166 6007
30 x 40 cm	129 0527
18 x 43 cm	146 3116
35 x 35 cm	164 0820
35 x 43 cm	190 1909

### Film processing

Manual processing or automatic processing using standard and rapid (60-second) cycles.

### Film care and handling

#### Storage:

50 to 70° F (10 to 21° C),  
30 to 50% relative humidity

#### Safelight:

Kodak GBX-2 or equivalent,  
standard frosted 15-watt bulb  
at least 4 feet (1.2 meters)  
from film-handling surface/area

Health Imaging Division  
EASTMAN KODAK COMPANY  
Rochester, NY 14650

KODAK CANADA INC.  
Toronto, Ontario M6M 1V3  
CANADA

Outside the U.S. or Canada, please  
contact your local Kodak company.

[www.kodak.com/go/health](http://www.kodak.com/go/health)

